

Micrel Spike PN Set Update

Micrel unveils improvements to Spike PN Set (MCKM1.EE.148.x)

Distribution: UK

Effective Date: 21st January 2016

Micrel has announced a number of improvements to the filter in the non gravity giving set (MCKM1.EE.148). The new filter used on the giving set is the same filter that is used by CME in the Bodyguard giving sets. The ordering code for the product remains unchanged.

When priming the set, the pump has to be turned in the opposite way to the way it is now: please see the instructions below:

Hold the pump into the vertical position, the 2 tubes in the upward position, ready for priming as shown below:



Press the 'Purge' soft key to prime the set with the pump. Repeat the operation if necessary until the PN solution exits from the tip of the Set/Extension line

Each carton of sets will include an instruction leaflet clarifying the revised priming process.

If you would like further information about this or any product in the Inspiration Healthcare range, please contact us on +44(0)1455 840 555 or by email on info@inspiration-healthcare.com

Moira Kallis RGN
Product Specialist